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Qvintip is a self-sampler with a detachable tip, that allows you to collect a vaginal fluid sample yourself and send it to laboratory for HPV (human papillomavirus) analysis.

The kit includes a self-sampler and a sample transport tube.

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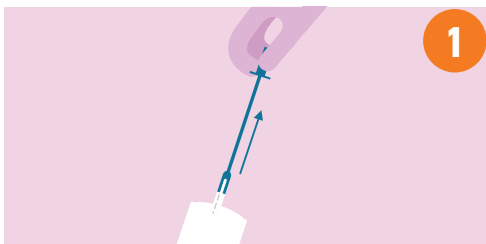
IMPORTANT NOTES!


- ✓ Qvintip should be handled with care and only according to these instructions.
- ✓ The white tip may not be removed or bent before sampling.
- ✓ Do not use if the packaging is damaged or if the expiry date has passed.
- ✓ Do not use during menstruation.
- ✓ Do not use during pregnancy or for three months following pregnancy.
- ✓ It is advisable to refrain from intercourse for 24 hours prior to sampling.
- ✓ Do not use any local vaginal treatment for at least two days before using the self-sampler.
- ✓ Vaginal contraceptives, condoms and water-based lubricants can be used as normal.
- ✓ For single use only. Re-use can result in infection and/or incorrect diagnosis.

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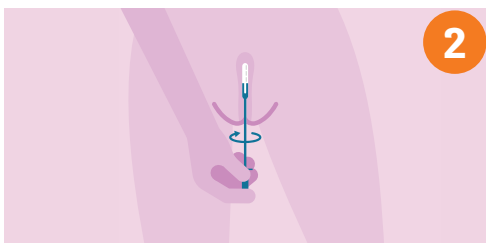Open the packaging and remove Qvintip sampler by holding the blue handle. Keep the sampler straight. Be careful not to touch anything with the white tip.

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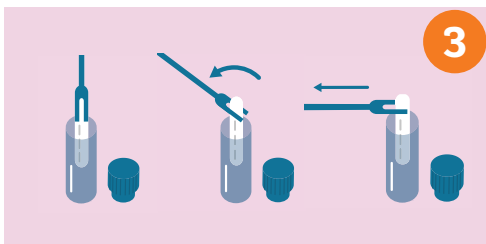
Slide the sampler gently into your vagina until you feel that it has reached the bottom (about 10 cm).

a) Turn the sampler five full rotations.


b) Remove the sampler (keeping it straight).

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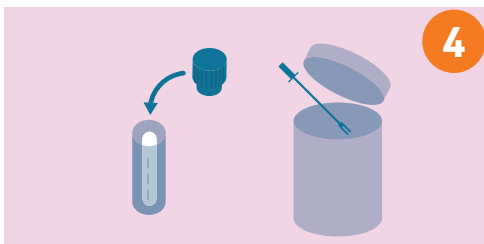
Hold the sampler in one hand and the tube in the other. Place the white tip of the sampler into the tube as far down as the figure shows. Bend the sampler until the blue part touches the tube. Slowly pull the handle away from the tube so that the white tip breaks off and falls into the tube.

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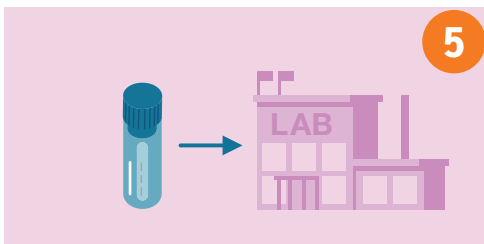Put the cap on the tube. Dispose the blue handle. The handle can be sorted as plastic.

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Place the tube into the plastic bag provided and seal it. Place the sample in the return envelope to send it via SmartPost parcel terminal **at the first available opportunity after collecting the sample**.

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To send the parcel, please go to your nearest SmartPost Itella parcel terminal. Select the option on the screen "Business client" > enter the door code **6758383** > open the door, place the envelope with sampling set in the cabinet and close the door. The parcel is on its way.